carl
marlin

FINEST DENTAL INSTRUMENTS

EASY
Clean

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INSTRUCTIONS FOR USE

REF 1953-ME/1953-IT

Zylinderampullenspritze • Cartridge syringes

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Description/application:

The Carl Martin cartridge syringe can be used for all standardized glass cartridges of 1.8 ml and cannula plugs with hole as well as with smooth surfaces. **⑦**

The syringe disposes replaceable tips in order to activate different cannula plugs. The head of the syringe has got a divisible outlet whereby a complete hygienic processing of the cartridge syringe is possible. **①②③**

Preparation of the injection unit by inserting the cartridge sideways. **④**

Therefore the plunger has to be pulled back with the screwed changeable tip, whereby a slight resistance of the spring at the end is noticeable. **⑤**

INSTRUCTIONS FOR USE

The cartridge is inserted ahead with the aluminium cap; the plunger with changeable tip runs against the cannula plug. According to the type of cannula plug, the following changeable tips are recommended for use: **⑥**

For hole plugs primarily the changeable tips 1 + 3 are used and for smooth cannula plugs tip no. 2. The locking of the changeable tips 1 + 3 in the hole plug is effected by a short and strong push by means of the plunger. When using smooth cannula plugs and the application of changeable tip no. 2, the clamps are pressed on the cannula plug and fastened in it by turning the plunger ring. After having fixed the changeable tip correctly, check if a tight connection has been created by pulling back the plunger.

INSTRUCTIONS FOR USE

Now the disposable cannula is screwed on the head of the syringe. After hosing out some drops of anaesthetic the cartridge syringe is ready to use.

After use on the patient, the ZY must be cleaned with suitable agents, disinfected and sterilized in accordance with QSA313 before further use.

The head can be unscrewed from the cartridge syringe and is divisible in individual parts. The needle outlet, which is divisible in two halves, is marked with A + B (even with several syringes of equal type, the pieces A + B always match). Furthermore, it is necessary to remove possible residues of anaesthetic inside the cartridge regularly by additional cleaning and visual inspection afterwards.

INSTRUCTIONS FOR USE

The changeable tips can be replaced by means of a special mounting key (s.P.12). Therefore please see the following graphic and textual descriptive procedure.

For dismantling the changeable tip proceed as follows:

1. Unscrew the changeable head (pic. 1 + 2)
2. Insert the mounting key (pic. 3)
3. Unfasten and unscrew the changeable tip (pic. 4)
4. Remove the changeable tip from the cartridge holder (pic. 5 - 7)

INSTRUCTIONS FOR USE

Below you receive an overview of all available components.

1953SP-SCHL
Mounting key

1953GK
Spare screw head

1953SP1
Changeable tip
„Arrow“

1953SP2
Changeable tip
„Craw“

1953SP3
Changeable tip
„Ball“

1953ME-SP
1953IT-SP
Changing head A+B

INSTRUCTIONS FOR USE

Syringe head
for needle

Changeable tip

Plunger

spring resistance

Plunger ring

QSA313
www.carlmartin.de

Carl Martin GmbH
Neuenkamper Str. 80-86
42657 Solingen-Germany

NEUENKAMPER STR. 80 - 86 · D-42657 SOLINGEN · GERMANY

TEL. +49(0)212/81 00 44 · FAX +49(0)212/8 73 45

E-MAIL: INFO@CARLMARTIN.DE · INTERNET: WWW.CARLMARTIN.DE