

WE TAKE CARE



SET
9020-EC



GOLDMAN-FOX
serrated
micro-serration 0.25



FINEST DENTAL INSTRUMENTS
GERMANY

over
100
years
since 1916

NEW 2019



EASY
Clean

A revolution in processing

The solution for a clean/residue-free processing.*

Residues below the hinge can be removed completely now by our newly developed **open2relock** system.

Easy – fast – clean



* In correspondence with required processing procedures according to EN ISO 17664



The basic material of the scissors consists of high quality stainless steel for medical purposes as usual.

The black coating is achieved by a so called PVD procedure (vacuum-based coating) that reaches a higher hardness. As a result, the durability of the product is significantly increased.

In addition, the dark colour reduces disturbing light reflections of the surgery lighting at the patient's chair.

The product is appropriated for all licensed disinfection and sterilization procedures (autoclave max. 134°C).


EASY CLEAN – the new definition for a perfect processing of your instruments.

Further products of the EASY CLEAN series can be found on our homepage

www.carlmartin.de

How to do it

For dismantling, open the scissors in the right angle – 90°

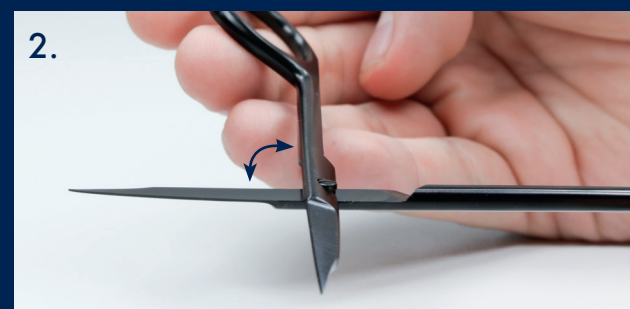

Lift the upper part of the scissors briefly (ca. 45°) and separate both parts from each other – DONE
(By appropriated handling, the scissors parts can be separated without great effort - otherwise risks of damages/break of the screw are not excluded.)



After processing has been effected, both scissors parts are put together again in reversed order (according to the pictures 1-2-3).

(In case of damages due to incorrect unlocking/reclosure we will not give any guarantee)