

The revolution in processing

B-0008-19-FL-1953 EasyClean 2019-EN



For 1.8 ml ampoules
ME = metric thread

1953ME-1.8-SET

Content:
1 x syringe
incl. changeable tip „Arrow“,
2 x changeable tip and changeable head
each



1953SP-SCHL
Mounting key



1953GK
Spare screw head



1953ME-SP
Changing head A+B



1953SP1
Changeable tip „Arrow“



1953SP2
Changeable tip „Craw“



1953SP3
Changeable tip „Ball“



1953ME-1.8

All displayed products are made of medical stainless steel!

**carl
martin**
Solingen



FINEST DENTAL INSTRUMENTS
GERMANY

over
100
years
since 1916

NEW 2019



**EASY
Clean**



NEUENKAMPER STR. 80 - 86 · D-42657 SOLINGEN · GERMANY
TEL. +49(0)212/81 00 44 · FAX +49(0)212/8 73 45
E-MAIL: INFO@CARLMARTIN.DE · INTERNET: WWW.CARLMARTIN.DE

We visualize hygiene!



By the constant return flow of the remaining anaesthetic of the needle after unscrewing, residues occur in the outlet of the head of the cartridge syringe.

A disinfection in this mostly encapsulated area is practically not workable.

The newly developed, DBGM* patented cartridge syringe with divisible syringe head enables a perfect processing now. Suitable for thermal disinfection and autoclave (max. 134°C). *DBGM = German federal utility model

To this you get the following illustrated information.

①



Standard cartridge syringe with fixed head (not unscrewable)

②



with unscrewable head - not dismantlable

③



NEW - unscrewable and dismantlable



New syringe head with divisible outlet. Hygienic processing and each time with new syringe head per patient. (Therefore we recommend to stock a sufficient number of syringe heads.)



Syringe head with fine thread for an easy removal.



Divisible needle outlet. By this a perfect processing can be effected. Both halves for the needle outlet are manufactured through computer-controlling (CNC) and marked with A + B each. With several syringes, A + B always match.



The cartridge syringe will be delivered with tip no. 1 as standard. Tips 1 + 3 are primarily recommended for hole plugs and no. 2 for solid plugs. The syringes and the necessary mounting key can be ordered separately if required.



Changeable tips for the fixation in the cannula plug (solid plug or hole plug)



For dismantling the changeable tip proceed as follows:
 1. Unscrew the changeable head (pic. 1 + 2)
 2. Insert the mounting key (pic. 3)
 3. Unfasten and unscrew the changeable tip (pic. 4)
 4. Remove the changeable tip from the cartridge holder (pic. 5 - 7)

