Colori

- Silicon Grip -

LiquidSteel meets Hi-Tech Silicon

**carl
marlin**
Solingen

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- Made of high-quality, thermostable special silicone
- Soft surface provides a comfortable gripping sensation
- Low weight
- Suitable for thermodesinfection and autoclave
- Available in 6 colours

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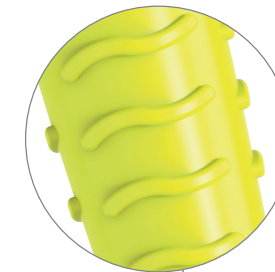
Put the main article number in front of the chosen colour.

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High-Tech UV laser marking allows a permanently durable inscription.

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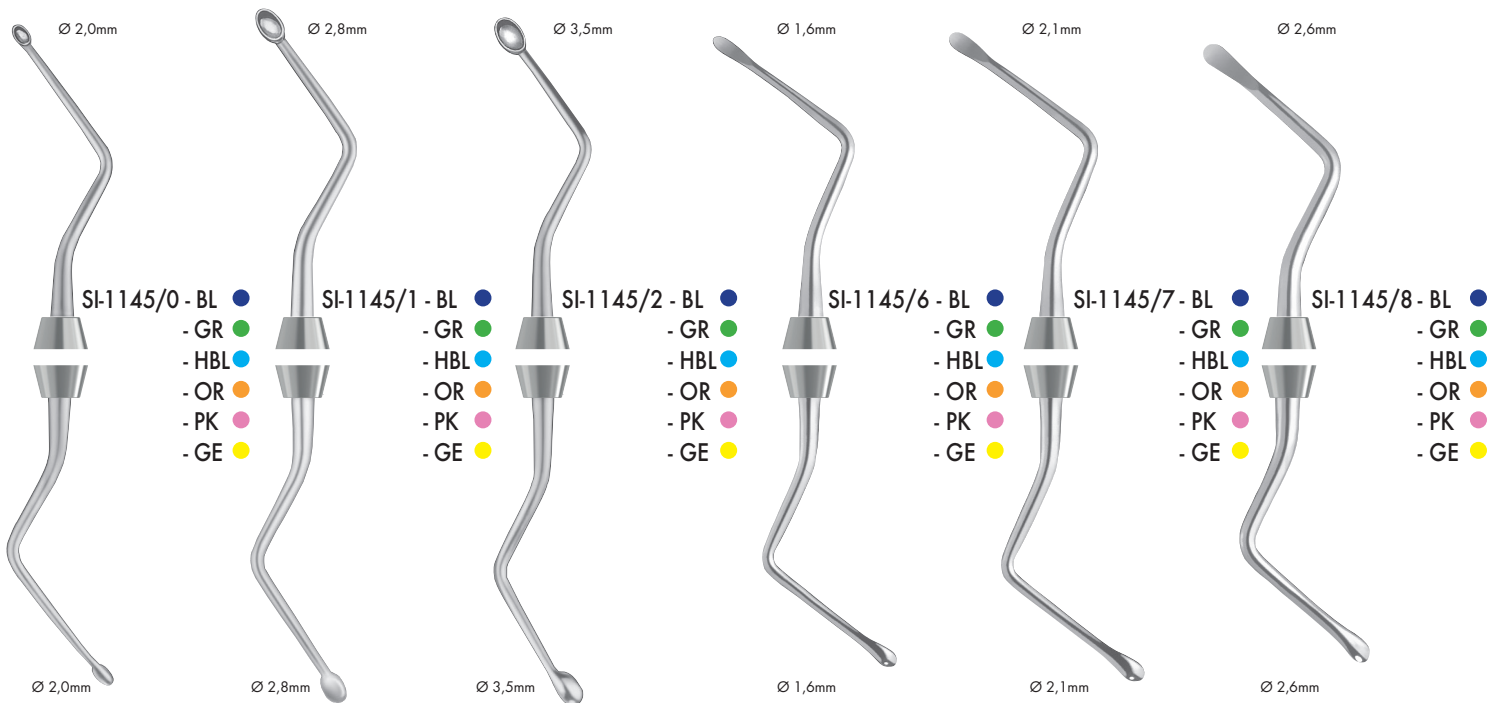
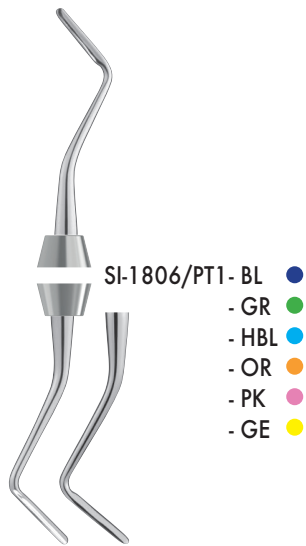
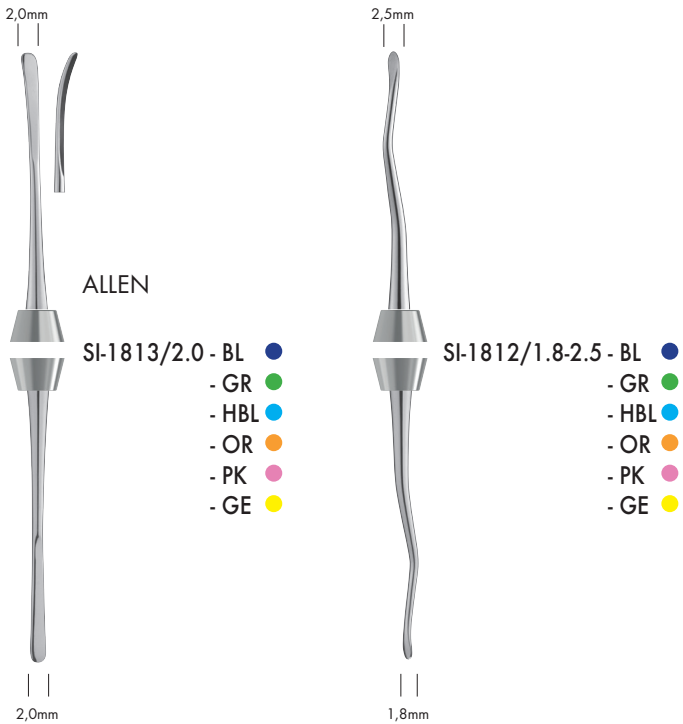
To avoid large flaps with the aim of a surgical undermining technique, these instruments are used for the creation of planar tunnelling incisions into the tissue. Such surgical procedures are for example indicated in the field of tissue augmentation using the tunnel technique.

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For reflecting and retracting the mucoperiosteum after incisions of the gingival tissue.

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
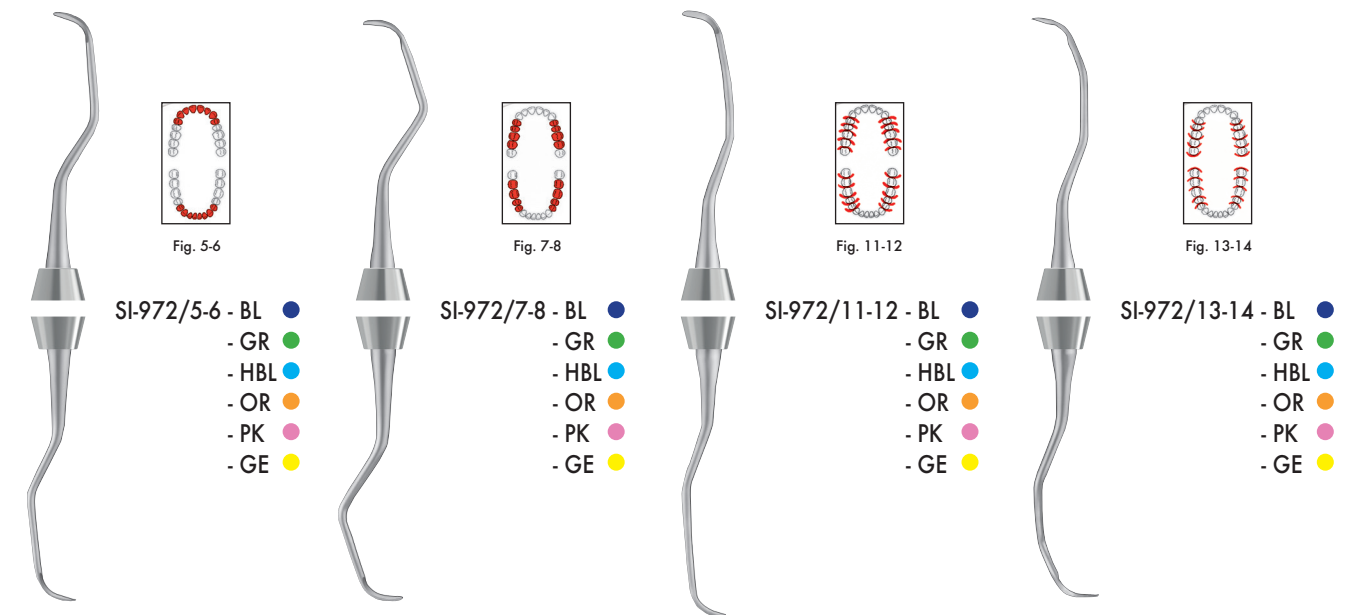
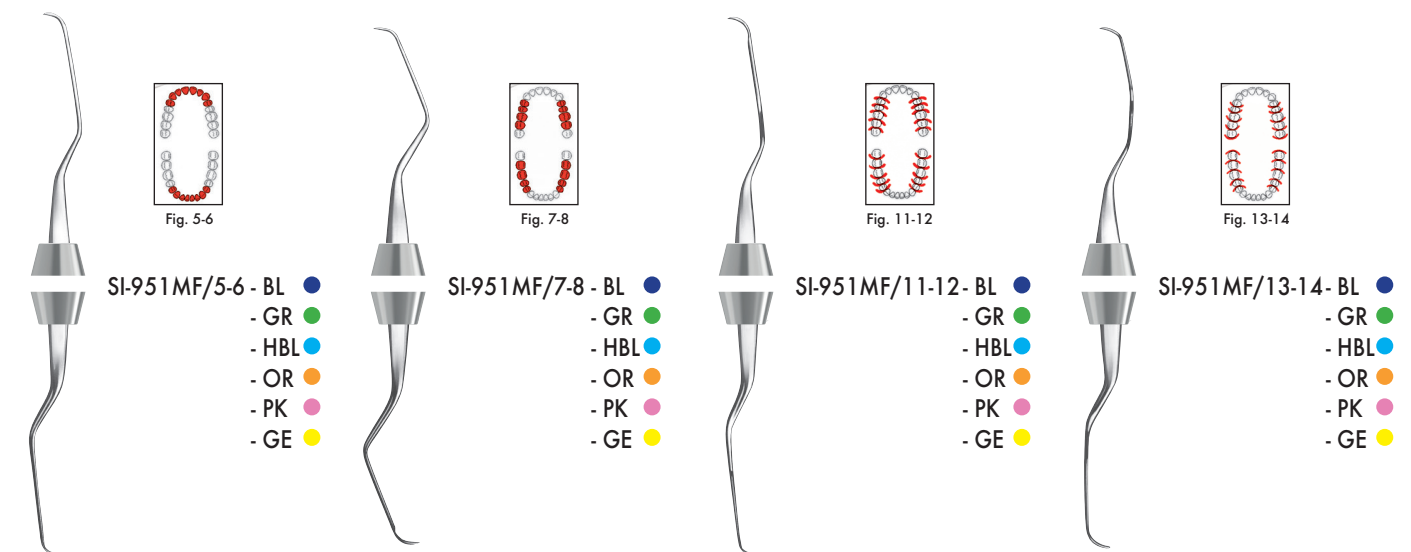
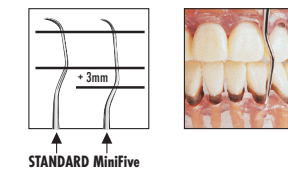
The 1st shaft 3mm longer but shorter working ends
than standard-GRACEY for periodontal pockets
at least 5mm deep. The short working ends are
especially adapted for small root surfaces (incisors)

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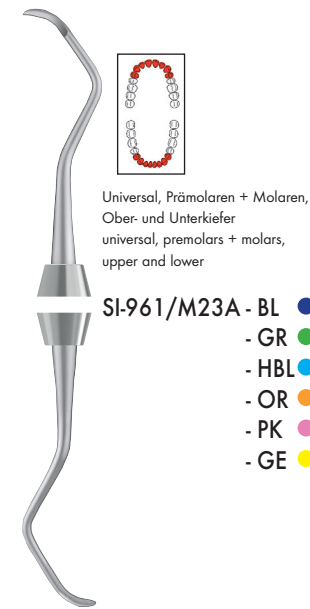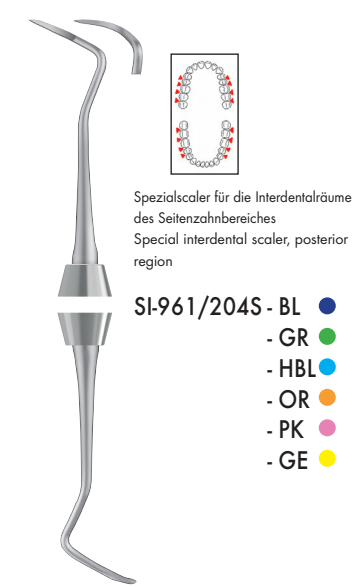
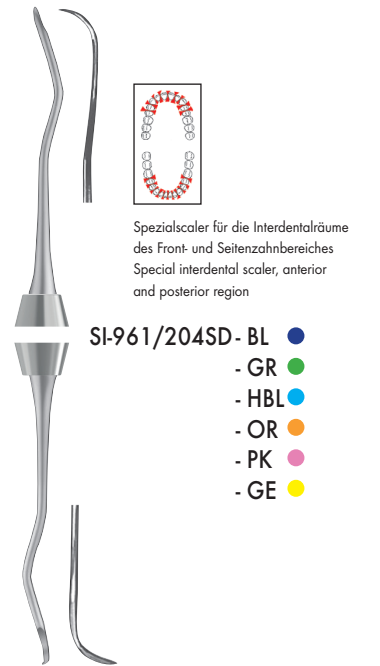
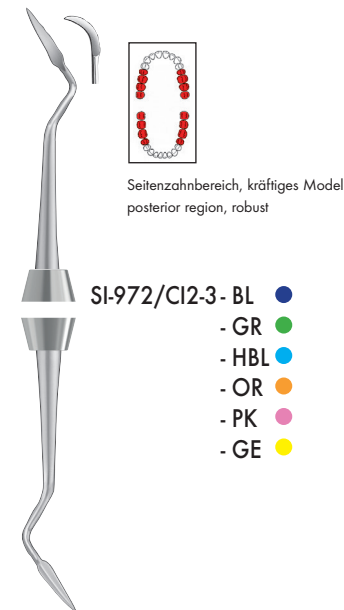
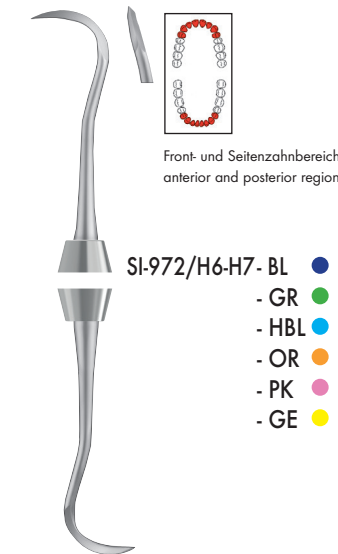
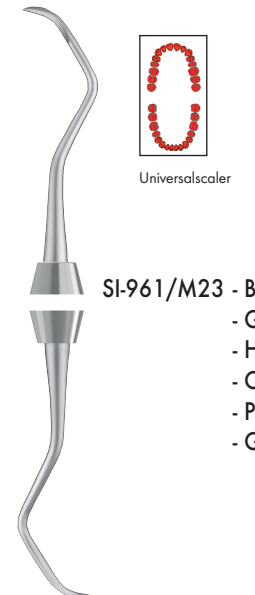
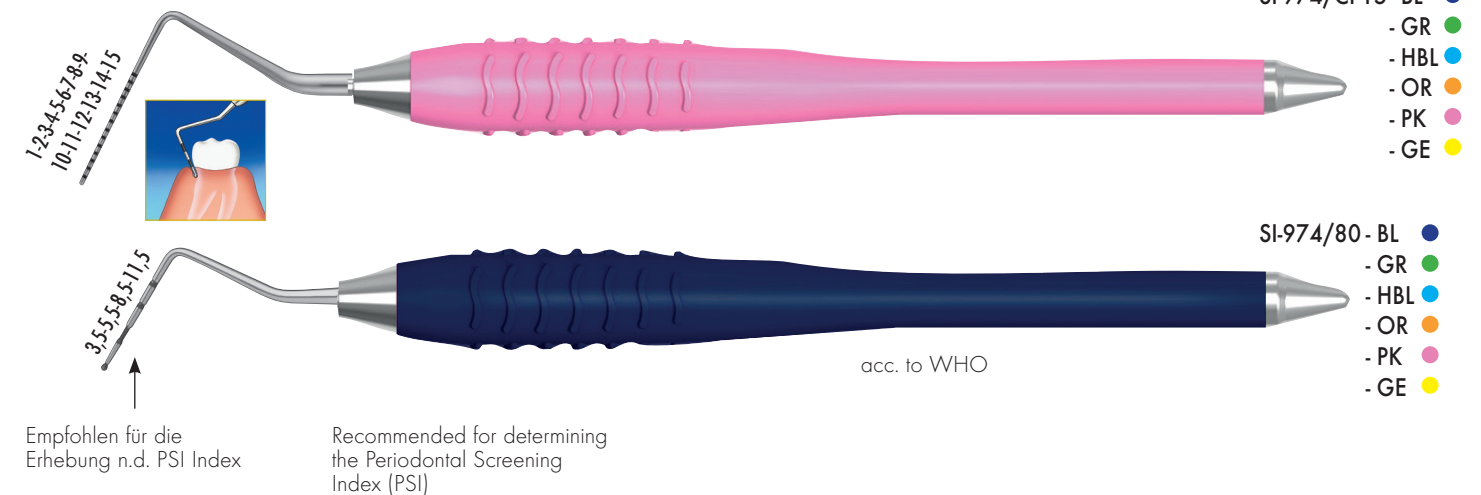
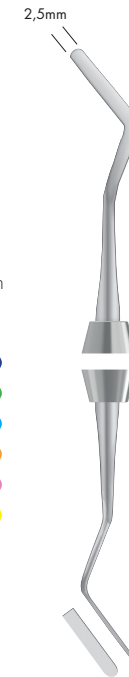
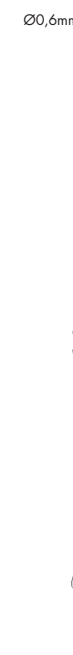
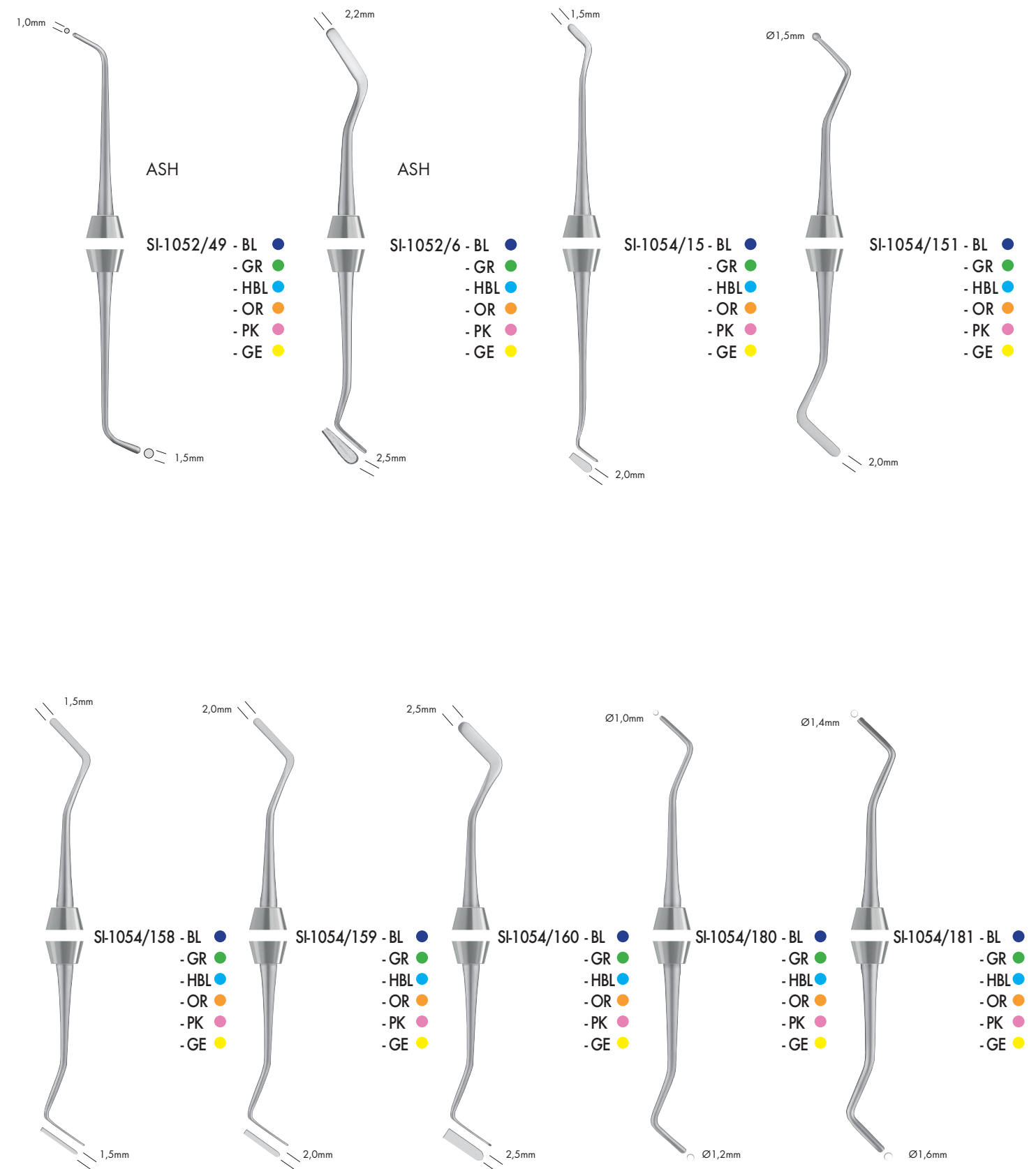
blue titanium oxide coating to reduce
the sticking of the composite

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SI-1051/108 - BL - GR - HBL - OR - PK - GE

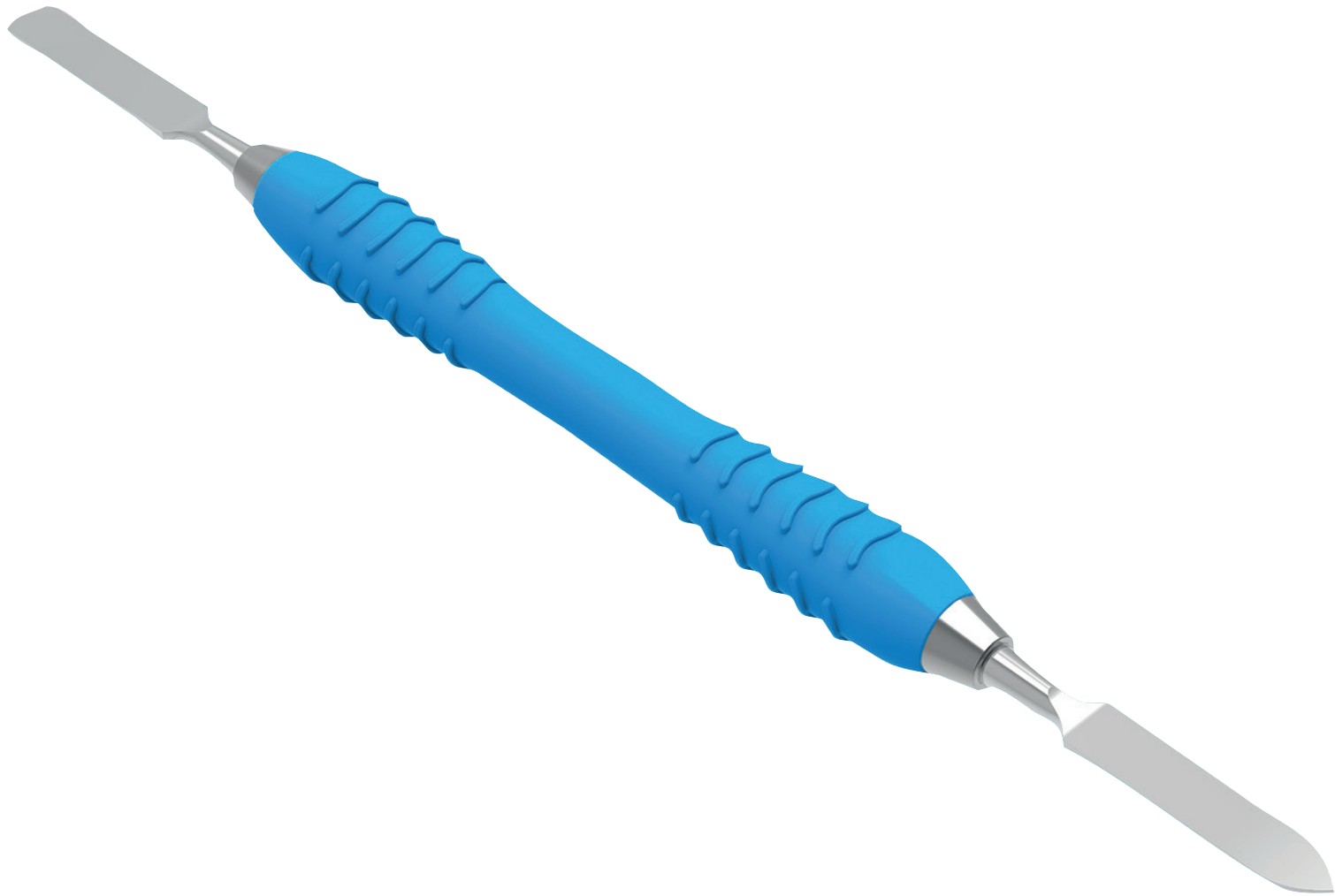
sehr feiner, flexibler Spatel
very fine and flexible spatula

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SI-1051/Carve - BL - GR - HBL - OR - PK - GE

Composite-SET
SI-1051-SET6
Inhalt/Content: je/each 1x SI-1051/14-HBL,
SI-1051/95-GE, SI-1051/104-OR,
SI-1051/108-GR, SI-1051SF/1-BL,
SI-1051/Carve-PK

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Preparation of resterilizable instruments with silicone handle acc. ISO 17764: 2004

1.) **Preparation at the operating site:**

- Remove coarse dirt (blood, tissue, medicine) from the instruments immediately after use.
- Place in cold water (below 40°C) immediately after use.
- Do not use hot water, which might cause fixation of residues.

2.) **Pre-cleaning**

- Instruments should be placed in cold water for at least 5 minutes.
- Remove residues with a soft brush.

3.) **Cleaning / disinfection / drying**

- Clean in the Thermo-Disinfector according to ISO 17764 and ISO 15883.
- Automatic Thermo-Disinfection under consideration of national requirements from the Standand A0-Wert.
- Automatic drying according to the cleaning and disinfection units as the circumstances require. Dry with lint-free cloth manually if necessary.

4.) **Functional testing/ Maintenance**

- Optical inspection for cleanliness.
- If necessary, repeat the above process until the instrument is visually clean.

5.) **Packaging**

- Choose packaging which is applicable to the instruments and the sterilization process. The sealing should not be performed under tension.

6.) **Sterilization**

- Sterilization in the autoclave at a temperature between 132°C and 137°C.
- Drying time: at least 10 minutes.

7.) **Storage**

- Store in a dry, clean and dust-free environment.
- The instruments must not be subjected to temperature fluctuations in order to minimize the risk of corrosion damage.

8.) **Information on the validation of the preparation:**

The following materials and machines were used during validation:

Thermodesinfektor:	MIELE PG 8582
Cleaning agents:	Dr. Weigert GmbH & Co. KG - neodisher MediClean forte
Neutralizer:	Dr. Weigert GmbH & Co. KG - neodisher Z

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Validation of the processing instructions according to DIN EN ISO 17764 by ValiTech GmbH & Co. KG (accredited by the German Accreditation Body according to DIN EN ISO / IEC 17025: 2005)

The validation proves that the instruments can be prepared in accordance with DIN EN ISO 15883 with a standard cleaning and disinfecting unit.

**carl
martin** 
Solingen

FINEST DENTAL INSTRUMENTS
- SINCE 1916 -
GERMANY

NEUENKAMPER STR. 80 - 86 · D-42657 SOLINGEN · GERMANY

TEL. +49(0)212/81 00 44 · FAX +49(0)212/8 73 45

E-MAIL: INFO@CARLMARTIN.DE · INTERNET: WWW.CARLMARTIN.DE