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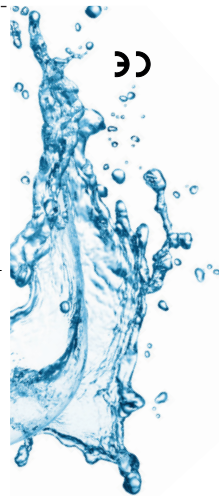
**INSTRUMENT.CARE**

FINEST DENTAL INSTRUMENTS

## Instructions for cleaning and sterilisation

### 1) Use and notes

The photo mirrors are used for intraoral images with indirect vision. Select the shape and size of the mirror according to the area to be photographed and the morphology and behaviour of the patient.

Types of mirrors	Large mouth	Small mouth	Area		
			Palatal	Lateral	Lingual
464/1	X		X		
464/2		X	X		
464/3		X		X	
464/4	X	X			X
464/5	X	X			X
464/6	X	X		X	

#### • Caution!

If the patient has problems with restricted opening of the mouth, the photo mirrors may cause stress (e.g. by retention of the tongue).

#### • Do not use if...

...the photo mirrors are cracked or scratched, as a good photographic result cannot be guaranteed.

...the photo mirrors are fractured or overused, as they could break in the patient's mouth.

### 2) Composition

The front side of the photo mirrors is made of pure titanium-coated float glass.

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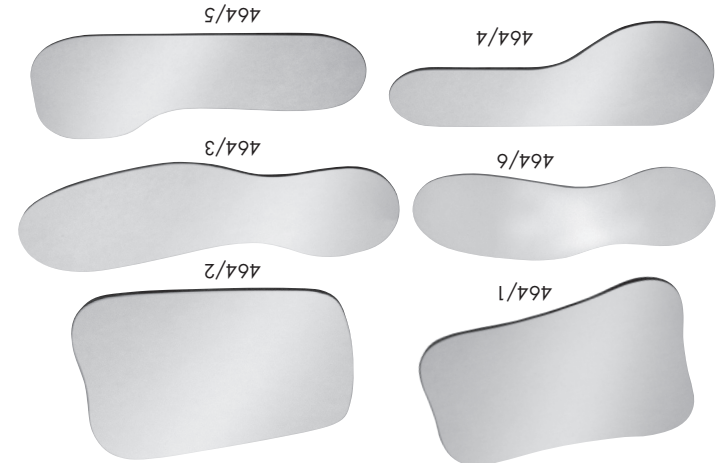
051 173 Rev. 8 16.07.2018

### 3) Decontamination, cleaning and sterilisation

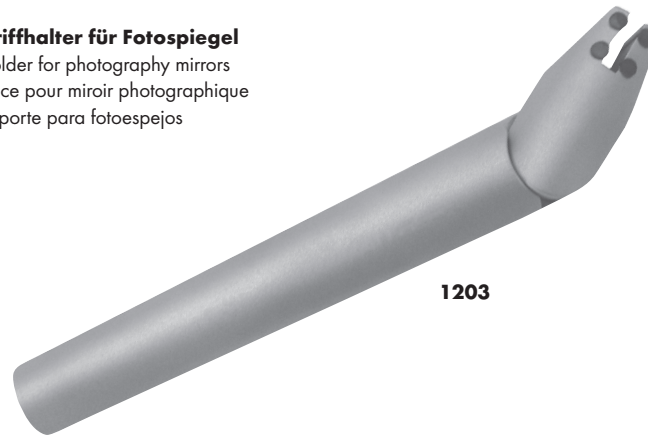
Stages	Process	Note
3.1 Cleaning & Disinfection - manual -	Immersion in a bath with disinfection solution	Observe the manufacturer's instructions
3.2 Cleaning & Disinfection -automated-	Ultrasonic bath Thermal disinfectant	The mirror should be placed in a separate bath to avoid damage from other instruments. It is recommended to disinfect the mirrors in a suitable holder to avoid possible damage caused by movement within the machine. Use the Carl Martin photo cassette for this purpose.
3.3 Rinsing & Drying	Only rinse with distilled water and then dry. Use a drainage system	Do not use abrasive products for drying
3.4 Visual inspection	Inspect and, if necessary, sort out mirrors that can no longer be used in perfect condition	Check the surface and the edges, as chips may occur and the mirror could therefore break during use.
3.5 Preparation for sterilisation	Pack the mirror in a bag or in a suitable sterilisation container	Use the Carl Martin photo cassette for this purpose. Art. No. 3014
3.6 Sterilisation	Autoclave 134° Chemical 132° Drying 180°	Select a sterilisation cycle suitable for the mirrors.

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